


Woven Elastic Knee Support with Spiral Stays

EXPRESS

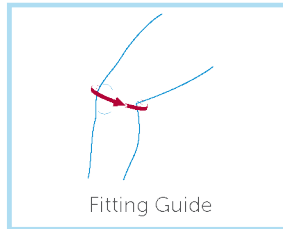

Indications

Mild Instabilities, Compression and warmth, OA/RA, Sports use post injury.

Fitting Instructions

- Please select correct size support using the measurements provided.
- Simple pull on design.
- Pass foot into the support with the patella marking at the front and pull up over the knee.
- Patella must be aligned in the centre of the brace
- Spiral stays should be positioned either side of the knee.
- Ensure the brace is not too loose or tight.

Care Instructions

- The brace can be hand or machine washed at low temp.
- Dry the garment naturally away from a direct heat source.
- Contact your issuer for a replacement as required.



Any orthosis purchased from Orthotix and fitted by anyone other than a qualified healthcare professional does so at their own risk. Orthotix is not liable for any damages incurred through ill-fitted or inappropriate product, if in doubt you should contact a suitable professional for further advice.



OD011/24 Rev 0

T: 02920 370696

E: enquiries@orthotix.co.uk

www.orthotix.co.uk

ORTHOTIX

SPORTS & ORTHOPAEDIC BRACING

Supporting your wellbeing

Official Supplier To:



Please visit our webshop to purchase our products direct:

www.orthotix.co.uk

